



*GlucoSure*TM **ADVANCE Link**

Blood Glucose Monitoring System

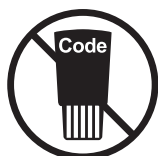

Autocode

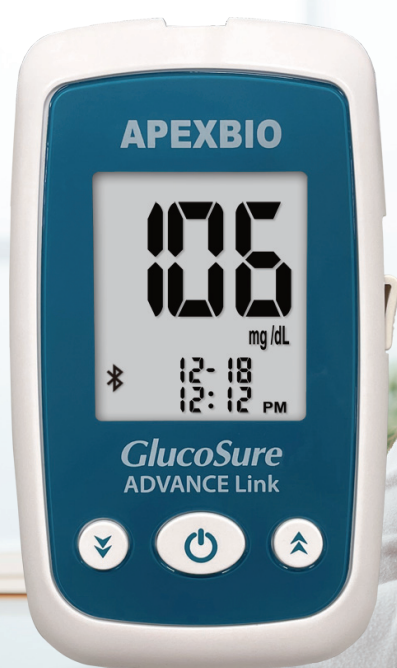


Small Blood Sample



Fast Result



Strip Ejection

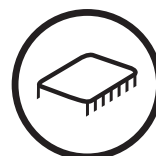

Large Memory



Bluetooth

The GlucoSure Advance Link Blood Glucose Monitoring System is designed to meet stringent accuracy requirements while remaining highly user-friendly and easy to operate. Its advanced [FAD]GDH enzyme system and proprietary algorithm, combined with innovative sensing technology, enable the GlucoSure Advance Link System to deliver highly accurate blood glucose results. Bluetooth connectivity provides the convenience of wirelessly uploading test results for seamless data management.

APEXBIOTM

GlucoSureTM ADVANCE Link Blood Glucose Monitoring System

Easy to use



1. Easy Insertion



2. For Fingertip Testing



3. Apply Sample



4. Results in 8 Seconds /
Data Transmission via BLE

Specifications

Blood sample: Capillary whole blood

GDH-FAD

Measurement Range: 20-600 mg/dL (1.1-33.3 mmol/L)

Test Time: 8 Seconds

Sample Volume: 0.8 µL

Hematocrit Range: 10-70%

Memory: 700 Test Results

Data Transfer: Bluetooth

Battery: 2 x CR 2032 3V Lithium Coin Cell Batteries

Dimensions: 105L x 60W x 14.5H (mm)

Weight: 51g without Battery

Accuracy

Table 1: Accuracy for blood glucose level < 75 mg/dL

Sample source	Within ± 5 mg/dL	Within ± 10 mg/dL	Within ± 15 mg/dL
Lay users Fingertip vs. YSI	10/11 (91%)	11/11 (100%)	11/11 (100%)
Lay users Palm vs. YSI	8/11 (73%)	10/11 (91%)	11/11 (100%)
Lay users Forearm vs. YSI	9/11 (82%)	10/11 (91%)	11/11 (100%)

Table 2: Accuracy for blood glucose level ≥ 75 mg/dL

Sample source	Within ± 5%	Within ± 10%	Within ± 15%	Within ± 20%
Lay users Fingertip vs. YSI	72/126 (57%)	115/126 (91%)	125/126 (99%)	126/126 (100%)
Lay users Palm vs. YSI	78/126 (62%)	114/126 (90%)	125/126 (99%)	126/126 (100%)
Lay users Forearm vs. YSI	72/126 (57%)	115/126 (91%)	125/126 (99%)	126/126 (100%)

Linear equation of each sample

Fingertip	$y = 0.9816x - 1.3045$
Palm	$y = 0.9895x - 2.741$
Forearm	$y = 0.981x + 2.3798$



* ©2026 Apex Biotechnology Corp. All rights reserved. APEXBIO and all product logos are registered trademarks of Apex Biotechnology Corp.

* Specifications may vary according to different authorized bodies.

* Actual content of the product is subject to change without notice.

